### Mr. Clear™ Anti-Fog

#### **EN - INTENDED USE:**

Key Surgical® Mr. Clear™ Anti-Fog is intended for use during endoscopic, laparoscopic, gastroscopic, and arthroscopic procedures, as well as any other procedures which require the use of an endoscope device, to prevent fogging of the endoscope lens.

#### **CONTRAINDICATIONS:**

There are no known contraindications and/or adverse effects.

#### **PRECAUTIONS:**

This device is single use only; reuse may lead to patient cross contamination. Do not re-sterilize. Re-sterilization of the device may result in loss of product functionality. The solution should be used in accordance with the instructions for use and any contraindications, warnings or precautions provided by the manufacturer of the associated instrument.

#### **CAUTION:**

When using this device with other devices, follow all the instructions and safety warnings that come with that device. Prior to use visually inspect packaging. Do not use if damaged, opened or breached as sterility cannot be guaranteed. USA Federal law restricts this device to sale by or on the order of a physician.

#### **INSTRUCTIONS FOR USE:**

Dispense pouch contents onto sterile field.  
Remove paper backing from foam pad and place foam onto sterile drape.  
Apply solution directly onto endoscope lens.  
Alternately, remove cap and apply solution onto foam pad. Touch scope lens to foam pad leaving a thin film.  
Reapply as needed.

#### **DISPOSAL:**

Mr. Clear Anti-Fog should be disposed of after use; in accordance with facility standard practices.

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