

Bite Blocks

INTENDED USE:

Key Surgical® Bite Blocks are intended to ensure patient's mouth remains open, with teeth protected, during endoscopic procedures and to protect the endoscope.

Product Number	Description	Port Size
EN-320	Adult bite block w/ VELCRO® brand strap	21 mm / 60 FR
EN-310	Adult bite block w/ pliable strap	21 mm / 60 FR
EN-210	Pediatric bite block w/ pliable strap	16 mm / 48 FR
EN-315	Adult O ₂ -CO ₂ bite block w/ pliable strap	21 mm / 60 FR
EN-300	Comfort block (no strap)	16 mm / 48 FR

CONTRAINDICATIONS:

No known contraindications and/or adverse effects.

INSPECTION:

Inspect all inner and outer surfaces for cracks and sharp edges.

Do not use if product is damaged or is not in original individual packaging.

INSTRUCTIONS FOR USE:

1. Place bite block in the patient's mouth.
2. If retaining strap is used, secure the strap on the bite block.
 - a) Pliable strap (EN-210, EN-310, EN-315): To secure the retaining strap, thread over cleat on side of bite block.
 - b) VELCRO® brand strap: (EN-320): Press strips together to fasten and pull apart to unfasten.
 - c) If no strap is used (EN-300), hold the bite block in place with pressure from hands or fingers during the entire procedure.

DISPOSAL:

Dispose after use according to facility protocol. Do not reuse.



MANUFACTURER: KEY SURGICAL LLC
8101 WALLACE ROAD / EDEN PRAIRIE, MN 55344 / USA
PH: 800.541.7995 KEYSURGICAL.COM



CATALOG
NUMBER



DO NOT RE-USE



LOT
NUMBER