## Cautery Tip Cleaners

### **[EN] INTENDED USE:**

Cautery Tip Cleaners are intended for use to remove eschar buildup from the tips of electrosurgical cauterization blades during surgical procedures.

### **PRECAUTIONS:**

Consult the instrument manufacturer's IFU to ensure the abrasive pad is indicated for use to clean the instrument.

This device is single use only; reuse may lead to patient cross contamination and/or device failure.

Do not re-sterilize. Re-sterilization of the device may result in loss of product functionality.

### **CAUTION:**

USA Federal law restricts this device to sale by or on the order of a physician.

Prior to use visually inspect packaging. Do not use if damaged, opened or breached as sterility cannot be guaranteed.

### **INSTRUCTIONS FOR USE:**

1. Remove sterile cautery tip cleaner pad from packaging.
2. Remove backing to expose adhesive and apply to appropriate flat surface.
3. Place instrument tip on pad to clean.
4. Apply pressure in a downward motion and do not allow insulation area of the instrument to come in contact with the grit; which can compromise the insulation.
5. Repeat until surgical instrument tip is visibly free of tissue and debris.
6. Check insulation to make sure it is still intact.

### **DISPOSAL:**

Dispose of cautery tip cleaner pad after use. Dispose of pad in compliance with facility guidelines for disposing of biohazard waste.

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