Scope Brushes (Stainless Steel Handle, Single Use)

EN - INTENDED USE:

Key Surgical® Scope Brushes are intended to be used in the manual cleaning of endoscopes. The brushes are used to clean the inside of the endoscope channel(s) and loosen soil present.

PRECAUTION:

Single use only; reuse may lead to patient cross contamination.

CONTRAINDICATIONS:

Do not use these brushes for any purpose other than their intended use.

Do not use reusable cleaning brushes for cytological tissue sampling or any other diagnostic or therapeutic purposes. Patient injury, cross-contamination and equipment damage may occur.

PREPARATION:

Does not need to be cleaned or disinfected prior to use. Can be put through a washer if desired.

INSPECTION:

Inspect and replace if worn, frayed, bent or otherwise damaged. Worn or damaged bristles and handles are ineffective in cleaning and may damage the endoscope.

INSTRUCTIONS FOR USE:

1. Ensure brush is the proper diameter and length for the endoscope channel it is being used to clean.
2. Advance the brush in short strokes to avoid kinking of the handle.
3. Brushes should not be coiled in loops smaller than 10" (25cm).
4. To remove gross soil from the scope after brushing, flush the scope with water until the soil is removed.
5. For specific scope cleaning instructions, refer to the scope manufacturer's IFU.

STERILIZATION:

This sterilization is intended for one time use of brushes in a sterile setting and is not for brushes that have already been used.

New, unused brushes are provided non-sterile and have been validated for sterilization efficacy using steam sterilization according to the following parameters:

Steam Sterilization		
Cycle Type	Temperature	Minimum Time –Full Cycle
Gravity	121° C(250° F)	30 minutes
Pre-Vacuum	132° C(270° F)	4 minutes

DISPOSAL:

Dispose after use according to facility protocol. Do not reuse.

WARNING:

Serious incidents that have occurred in relation to this medical device should be reported to the manufacturer and competent authority in the country where the incident occurred.

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KEY SURGICAL LLC
MDSS UK RP LTD