## Disposable Corner Protectors (Card / Paper)

### EN - INTENDED USE:

Key Surgical® Tray Corner Protectors are intended for use during sterilization, organization and storage of surgical trays.

### CONTRAINDICATIONS:

No known contraindications and/or adverse effects.

### PRECAUTIONS:

Visually inspect the devices before use to ensure that they are free from visible contaminants or any damage such as rips or tears.

Store away from any sources of ignition.

Like any paper product, handle in such a way as to prevent cuts.

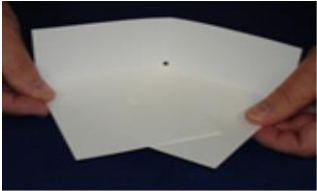
Single-use only; reuse may lead to patient cross-contamination.

### INSTRUCTIONS FOR USE:

1. Does not need to be cleaned or disinfected before initial use
2. Fold, tuck, and pinch to create a corner for the trays/baskets.
  - a. Fold


- b. Tuck

- c. Pinch

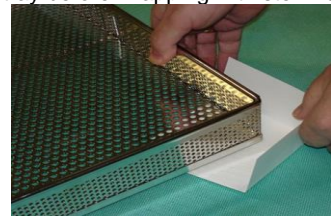3. Place the corner protectors on the top and bottom corners of the tray before wrapping with sterilization wrap.

4. Wrap the tray according to standard wrapping procedures. Appropriate sterilant contact will occur on the surface of the corner protectors as well as the tray surface under the corner protectors.

### STERILIZATION:

Evaluated for sterilization compatibility and found to be compatible with the following parameters:

Steam Sterilization		
Cycle Type	Temperature	Minimum Time – Full Cycle
Pre-Vacuum	134° C (273° F)	3.5 minutes
Pre-Vacuum	134° C (273° F)	18 minutes

It remains the responsibility of the processor to ensure that the processing, as actually performed using equipment, materials and personnel in the processing facility, achieves the desired result. This requires verification and/or validation and routine monitoring of the process.

### DISPOSAL:

Used: Dispose after use according to facility protocol. Do not reuse.  
Unused: It is suitable for recycling in the wastepaper stream.

### WARNING:

Serious incidents that have occurred in relation to this medical device should be reported to the manufacturer and competent authority in the country where the incident occurred.

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KEY SURGICAL LLC
MDSS UK RP LTD