



Acetabular Reamer Brushes

INTENDED USE:

Key Surgical® Acetabular Reamer Brush is intended to clean dirty acetabular bone reamers.

CONTRAINDICATIONS:

There are no known contraindications and/or adverse effects.

PREPARATION:

Brushes are reusable and must be cleaned and disinfected prior to reuse.

CLEANING & DISINFECTION:

Clean and disinfect brushes daily, at a minimum, prior to reuse. Brushes can be cleaned manually or using an automatic washer/disinfector.

INSPECTION:

Brushes should be inspected between uses and replaced when worn, frayed, bent or otherwise damaged. Worn or damaged bristles and handles are ineffective in cleaning, and may damage the device.

STERILIZATION:

This sterilization is intended for one time use of brushes in a sterile setting, and is not for brushes that have already been used.

New, unused brushes are provided non-sterile and have been validated for sterilization efficacy using gravity and pre-vacuum steam sterilization according to the following parameters:

Cycle Type	Temperature	Minimum Time – Full Cycle
Gravity	121° C (250° F)	30 minutes
Pre-Vacuum	132° C (270° F)	4 minutes
Pre-Vacuum	134° C (273° F)	3 minutes


Due to the steam sterilization process, there may be a slight discoloration of the nylon brush bristles; however, this does not impair the fit, form, or function of the brush.

STORAGE:

Store brushes in a manner that will reduce cross-contamination.

DISPOSAL:

There are no special disposal instructions.

 MANUFACTURER: KEY SURGICAL LLC
8101 WALLACE ROAD / EDEN PRAIRIE, MN 55344 / USA
PH: 800.541.7995 KEYSURGICAL.COM

 REF CATALOG NUMBER

 LOT NUMBER



 CONSULT INSTRUCTIONS FOR USE

 EC  REP MDSS GMBH
SCHIFFGRABEN 41, D-30175 / HANNOVER, GERMANY