

Towel & Utility Clamp

INTENDED USE:

Key Surgical® Towel & Utility Clamp is an accessory intended for use by medical personnel to hold and secure tubing, cords, and drapes.

The clamps are non-sterile, radiopaque, and not made with natural rubber latex.

INSTRUCTIONS FOR USE:

1. The towel & utility clamp must be cleaned or disinfected prior to initial use.
2. Sterilize when applicable.
3. The towel & utility clamp is reusable and must be cleaned, decontaminated, and/or sterilized prior to reuse according to the instructions below.
4. The towel & utility clamp is not intended for implantation or to come in contact with a patient or open tissue.

POINT OF USE CARE:

Clean clamp as soon as possible after use. If cleaning must be delayed, immerse in an enzymatic solution or water to prevent drying and encrustation of surgical soil. Remove excessive soil with a disposable wipe.

MANUAL CLEANING:

1. Pre-rinse under cold tap water for one (1) minute to remove gross debris.
2. Soak for a minimum of two (2) minutes in a pH neutral detergent, prepared in accordance with the manufacturer's instructions for use.
3. Rinse under cold tap water for one (1) minute.
4. Ultrasonically clean for a minimum of five (5) minutes in a neutral pH detergent, prepared in accordance with the manufacturer's instructions for use.
5. Rinse under cold tap water for one (1) minute.

AUTOMATED CLEANING:

It may be necessary to manually clean prior to automated processing to improve the removal of adherent soil. Follow the previous instructions for manual cleaning.

1. Run the automatic wash cycle – minimum cycle parameters:
 - 1 minute cold pre-rinse
 - 5 minute enzyme wash at 43° C (109° F) minimum temperature
 - 1 minute cold rinse
 - 7 minute dry at 90° C (194° F) minimum temperature

CLEANING INSPECTION:

Visually inspect before sterilization or storage to ensure the complete removal of soil from surfaces. If soil is still present, reprocess the clamp.

STERILIZATION:

The clamp has a life span and will require replacing if there are any signs of deterioration or loss of functionality. The clamp is provided non-sterile and has been validated for sterilization according to applicable international process standards and guidance for the following methods and parameters:

<i>Cycle Type (Steam)</i>	<i>Temperature</i>	<i>Minimum Time – Full Cycle</i>
Gravity	121° C (250° F)	30 minutes
Pre-Vacuum	132° C (270° F)	4 minutes

STORAGE:

Store the towel & utility clamp in a manner that will reduce cross-contamination.

DISPOSAL:

Dispose of product following facility policy for disposal.



MANUFACTURER: KEY SURGICAL LLC
8101 WALLACE ROAD / EDEN PRAIRIE, MN 55344 / USA
PH: 800.541.7995 KEYSURGICAL.COM

LOT

LOT
NUMBER

REF

CATALOG
NUMBER