## Silicone Clamp Covers

### **EN** - INTENDED USE:

Key Surgical® Silicone Clamp Covers are disposable, single-use accessories to medical devices intended for transient-term (less than 60 minutes) use in surgical procedures requiring grasping and clamping. The Silicone Clamp Covers are not intended to control, diagnose, monitor or correct a defect of the heart or of the central circulatory system through direct contact with these parts of the body and are not intended for use in direct contact with the central circulatory system or central nervous system.

### **PRECAUTIONS:**

This device is single use only; reuse may lead to patient cross contamination and/or device failure. Do not re-sterilize. Re-sterilization of the device may result in loss of product functionality.

### **CAUTION:**

Prior to use visually inspect packaging. Do not use if damaged, opened or breached as sterility cannot be guaranteed.

### **CONTRAINDICATIONS:**

There are no known contraindications associated with the Silicone Clamp Covers.

### **INSTRUCTIONS FOR USE:**

1. Remove silicone clamp covers from pouch.
2. Clamp covers may be trimmed to desired length.
3. Use during surgical procedure.

### **DISPOSAL:**

Upon completion of procedure, dispose of both used and unused clamp covers according to facility protocol. Do not reuse.

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KEY SURGICAL LLC