

INSTRUCTIONS FOR USE – ENDO LINER

ENDO-Liner is designed to protect reprocessed endoscopes from airborne contamination (particles and microorganisms) during transport and/or storage. Endo-Liner also protects patients, users and third parties from microbial contamination by used endoscopes on their way to reprocessing.

ENDO-Liner consists of four single use, sterile packed liners including one tray bag, one thin plastic tray liner and two colour-coded foils (red and green).

PREPARATION OF THE ENDOSCOPE

1. Open the Endo-Liner pack.
2. Open the transparent tray bag and insert the transport tray completely for complete protection.
3. Close the bag, using the adhesive tape and seal it. Ensure that all edges are enclosed around the tray.
4. Place the thin plastic tray liner in the tray to contain endoscope accessories and fluids left from the pre-cleaning stage. If not required, please dispose it.
5. Place the endoscope, accessories and the red foil inside the tray.
6. Cover the tray using the green foil for clear identification of clean endoscope.
7. The endoscope is now ready to be transported to the procedure room.

PATIENT PROCEDURE

1. Remove the green foil and dispose it.
2. Remove the endoscope and accessories.
3. After the endoscope has been used and pre-clean, place the endoscope and accessories back in the transport tray.
4. Cover tray using the red foil to clearly identify the endoscope as contaminated.
5. The endoscope and accessories are now ready to be safely transported to the decontamination unit.