



### General Instrument Cleaning Brushes

**INTENDED USE:**

Key Surgical® General Instrument Cleaning Brushes (BR-04-003, N-3000, N-4000 & S-4000) are intended to clean dirty instruments during the cleaning and decontamination process.

**CONTRAINDICATIONS:**

There are no known contraindications and/or adverse effects.

**CLEANING & DISINFECTION:**

Clean brushes after each use and disinfect between cases or daily, at a minimum. Brushes can be cleaned manually or using an automatic washer/disinfector.

**INSPECTION:**

Brushes should be inspected between uses and replaced when worn, frayed, bent or otherwise damaged. Damaged brushes cannot be cleaned effectively. Worn or damaged bristles and handles are ineffective in cleaning, and may damage the device.

**STERILIZATION:**

*This sterilization is intended for one time use of brushes in a sterile setting, and is not for brushes that have already been used.*

New, unused brushes are provided non-sterile and have been validated for sterilization efficacy using gravity and pre-vacuum steam sterilization methods according to the following parameters:

<i>Cycle Type</i>	<i>Temperature</i>	<i>Minimum Time – Full Cycle</i>
Pre-Vacuum	132° C (270° F)	4 minutes
Pre-Vacuum	134° C (273° F)	18 minutes
Gravity	121° C (250° F)	30 minutes

**STORAGE:**

Store brushes in a manner that will reduce cross-contamination.

**DISPOSAL:**

There are no special disposal instructions for brushes.

 MANUFACTURER: KEY SURGICAL, LLC  
8101 WALLACE ROAD / EDEN PRAIRIE, MN 55344 / USA  
PH: 800.541.7995 KEYSURGICAL.COM

**REF** CATALOG NUMBER

**EC** **REP** MDSS GMBH  
SCHIFFGRABEN 41, D-30175 / HANNOVER, GERMANY

**LOT** LOT NUMBER 