**Heat Protective Gloves (silicon)**  
**Hitzehandschuh aus Silikon**  
**09484, 09368**

**INTENDED USE:** Key Surgical® heat protective gloves provide high protection against moderate temperature up to 240° C/464° F (intended for use during removal of products from autoclaves/wash-disinfector).

**PRECAUTION:**

Perform visual inspection for usability and cleanliness of the heat protective gloves before use

**Note:** Damaged gloves no longer provide protection and must be replaced.

**APPLIED STANDARDS:**

Tested according to:

EN 407:2004-11, DIN EN 420:2010-03 (EN 420:2003+A1:2009)

**CLASSIFICATION:**

Category II per Regulation (EU) 2016/425 on personal protective equipment

**CLEANING AND MAINTENANCE:**

Mechanical reprocessing with or without ultrasound treatment:

**Cleaning and disinfecting chemicals and temperatures:** acidic / neutral / alkaline with or without surfactant additives, chemically at max. 60 °C / 140 °F or with demineralized water, thermally, at max. 93 °C / 199 °F. It is assumed that commercially available products approved for the intended application are used and that the user complies with recommended concentrations, exposure times and temperatures. Demineralized water must be used for the final rinse to ensure no residues are present post-reprocessing.

**Drying:** max. 100 °C / 212 °F

**DISPOSAL:**

There are no special disposal instructions.

**STORAGE CONDITIONS:** Store the product in a dry area away from direct sunlight.

Declaration of Conformity can be found at [keysurgical.com/resources](http://keysurgical.com/resources).

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**MANUFACTURER: KEY SURGICAL LLC**