

Magnetic Drapes

INTENDED USE:

Key Surgical® Magnetic Drapes are a sterile device intended to be used in a hands-free transfer of stainless steel surgical sharps and instruments during procedures.

PRECAUTIONS:

This device is single use only; reuse may lead to patient cross contamination and/or device failure. Do not re-sterilize. Re-sterilization of the device may result in loss of product functionality.

CAUTION:

Do not use product in MRI controlled environments. Product is magnetic and not MRI safe. Prior to use visually inspect packaging. Do not use if damaged, opened or breached as sterility cannot be guaranteed.

CONTRAINDICATION:

Caution should be used during procedures with materials which may be adversely affected by magnetic fields.

INSTRUCTIONS FOR USE:


1. Magnetic drapes are provided sterile. Remove drape from pouch and place on appropriate surface.
2. Place stainless steel surgical sharps and instruments on drape as needed for transfer during procedure.


STORAGE REQUIREMENTS:


Store at room temperature in dry conditions. Do not expose to direct sunlight.


DISPOSAL:


Dispose according to facility protocol. Do not reuse.







 KEY SURGICAL LLC
8101 WALLACE ROAD / EDEN PRAIRIE, MN 55344 USA
(US) TEL: 800.541.7995 (UK) TEL: +44(0)1628 810626
(EU) TEL: +49 4363 905900


 MANUFACTURER
FOR US: MANUFACTURED FOR


 DATE OF MANUFACTURE


 DO NOT USE IF PACKAGE
IS DAMAGED


 CONSULT
INSTRUCTIONS
FOR USE


 REF  CATALOG
NUMBER  LOT  LOT
NUMBER  STERILE  EO
STERILIZED USING ETHYLENE OXIDE


 SINGLE STERILE BARRIER SYSTEM


 SINGLE STERILE BARRIER SYSTEM
WITH PROTECTIVE PACKAGING INSIDE

 DO NOT RE-USE

 DO NOT RE-STERILIZE

 USE-BY DATE

 CAUTION

 **STERIS**®