Magnetic Drapes

EN - INTENDED USE:

Key Surgical® Magnetic Drapes are a sterile device intended to be used in a hands-free transfer of stainless steel surgical sharps and instruments during procedures.

PRECAUTIONS:

These instructions are for general use and application of the magnetic drapes. These instructions do not represent instructions for any procedures during which the products may be associated.

This device is single use only; reuse may lead to patient cross contamination and/or device failure.

Do not re-sterilize. Re-sterilization of the device may result in loss of product functionality.

CAUTION:

Do not use product in MRI controlled environments. Product is magnetic and not MRI safe.

Prior to use visually inspect packaging. Do not use if damaged, opened or breached as sterility cannot be guaranteed.

CONTRAINDICATION:

Caution should be used during procedures with materials which may be adversely affected by magnetic fields.

INSTRUCTIONS FOR USE:

1. Magnetic drapes are provided sterile. Remove drape from pouch and place on appropriate surface.
2. Place stainless steel surgical sharps and instruments on drape as needed for transfer during procedure.
3. This product is single use. Upon completion of procedure, dispose of used drape according to hospital protocol.

STORAGE REQUIREMENTS:

The magnetic drapes must be stored at room temperature in dry conditions. Do not expose the drapes to direct sunlight.

DISPOSAL:

When ready to be disposed of, dispose of magnetic drape according to hospital protocol.

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**MANUFACTURED FOR KEY SURGICAL LLC**