## Needle Counter with Blade Remover

### **EN – INTENDED USE:**

Key Surgical® Needle Counters are a single use, sterile device designed to safely facilitate correct count and disposal of used blades and needles during surgical procedures.

### **PRECAUTIONS:**

This device is single use only; reuse may lead to patient cross contamination and/or device failure.  
Do not re-sterilize. Re-sterilization of the device may result in loss of product functionality.

### **CAUTION:**

Do not use product in MRI controlled environments. Product is magnetic and not MRI safe.  
USA Federal law restricts this device for sale by or on the order of a physician.  
Prior to use visually inspect packaging. Do not use if damaged, opened or breached as sterility cannot be guaranteed.

### **CONTRAINDICATIONS:**

Caution should be used during procedures with materials which may be adversely affected by magnetic fields.

### **INSTRUCTIONS FOR USE:**

1. Remove adhesive backing on needle counter. Open needle counter and place on flat surface.
2. Place scalpel with used blade over the removal mechanism at an angle so the bottom left-hand corner of the blade is placed against the inner ridge of the removal mechanism. Slowly turn scalpel to the left until the used blade is secure within removal mechanism.
3. Ensure the tip of the used scalpel blade is pointing down and pull the scalpel handle to the right to begin the blade separation from the scalpel handle.
4. Slowly pull back the scalpel handle and pull to the right while keeping the used blade tip pointed down. The used blade will come off the scalpel handle and remain in the needle counter.

### **DISPOSAL:**

Dispose of needle counter in compliance with facility guidelines for disposing of used sharps.

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MANUFACTURED FOR KEY SURGICAL LLC