## Surgical Skin Markers

### **EN** - INTENDED USE:

Surgical Skin Markers are intended for use to mark anatomical sites prior to patient procedures by the health care professional on intact skin.

### **PRECAUTIONS:**

- Consider whether the patient is sensitive to gentian violet.
- Skin surface must be clean and dried before marking skin.
- This device is single use only; reuse of marker may lead to patient cross contamination and/or device failure.
- Do not re-sterilize. Re-sterilization of the device may result in loss of product functionality.
- Protect the tip when using the skin marker and cap after use to avoid the ink volatilizing.

### **CAUTION:**

Prior to use visually inspect packaging. Do not use if damaged, opened or breached as sterility cannot be guaranteed.

### **CONTRAINDICATIONS:**

Rulers packaged with skin markers are not measuring devices and should be used as an indicator only, and not for accurate measurements.

### **INSTRUCTIONS FOR USE:**

1. Remove sterile skin marker from packaging.
2. Remove the skin marker cap to expose the marker tip.
3. Mark the desired areas on intact skin with the skin marker according to facility protocol.
4. Allow the markings to dry thoroughly before prepping.
5. Prepare and apply prep solution as instructed by prep solution manufacturer. Markings may fade slightly but remain legible.
6. Remove markings with isopropyl alcohol if desired. Markings will disappear naturally with bathing in a few days.

### **DISPOSAL:**

Dispose of skin marker after use. There are no special disposal instructions.

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