



Autoclave Gloves

Instructions for Use

Intended Usage

SteriBest Autoclave Gloves are designed for removing hot items from autoclaves. These 100% cotton gloves are rated to 250 degrees F.

Use Instructions

1. Place hands all of the way into the gloves.
2. Ensure that the gloves are covering your hands completely.

Cleaning and Disinfection

Do not use any bleach or dry-cleaning solution when cleaning these gloves.

1. Use cold water to rinse the gloves.
2. Use lukewarm water if pre-soaking.
3. Wash with mild detergent in warm water with a temperature of up to 160°F for up to 15 minutes.
4. Rinse the gloves in cold water.
5. Repeat the above steps as often as necessary to thoroughly clean the gloves.
6. Tumble dry the gloves until dry. The temperature setting should not exceed 150°F.

Disposal

SteriBest Autoclave Gloves have no specific instructions for disposal.

IFU 2: Autoclave Gloves

Rev A: 20190401 IRM

RICA Surgical Products, Inc.

Phone: 1-800-889-3218 | Fax: 1-847-671-5690 | www.ricasurgical.com
9207 Ivanhoe Street | Schiller Park, IL 60176