

Stringer Assistant

INTENDED USE:

Key Surgical® Stringer Assistant is designed to assist in the stringing of surgical instruments onto instrument stringers.

CONTRAINDICATIONS:

No known contraindications and/or adverse effects.

PREPARATION:

Clean and disinfect stringer assistant as needed.

CLEANING & DISINFECTION:

The stringer assistant has been functionally validated with the following:

- Polystyca 2X Enzymatic
- Cidex OPA

INSTRUCTIONS FOR USE:

1. Slide stringer into slot on base of the stringer assistant.
2. String one instrument upon another by the ringed handles.
3. When full, close off stringer (where applicable) and slide it out of the base.

STORAGE:

Store in a manner that will reduce cross-contamination.

DISPOSAL:

Requires replacing if there are any signs of deterioration or loss of functionality. Dispose according to facility protocol.



KEY SURGICAL LLC
8101 WALLACE ROAD / EDEN PRAIRIE, MN 55344 USA
(US) TEL: 800.541.7995 (UK) TEL: +44(0)1628 810626
(EU) TEL: +49 4363 905900



MANUFACTURER
FOR US: MANUFACTURED FOR



LOT
NUMBER



REF
CATALOG
NUMBER

 **STERIS**®