



Tray Liners

INTENDED USE:

Key Surgical® Tray Liners are intended for placement on the bottom of a sterilization tray as a means of protection and to absorb wetness and humidity during processing.

CONTRAINDICATIONS:

There are no known contraindications and/or adverse effects.

PREPARATIONS FOR USE:

1. Select appropriate size liner(s) for tray.
2. Place liner(s) inside the wrapped tray or rigid container system. Liner(s) may be placed inside or under the tray.

Tray Liners can be used in the following configurations:

Pre-Vacuum Steam/100% Ethylene Oxide (EtO)

- Up to 2 tray liners inside and 2 tray liners outside
- Up to 8 folded layers (4 tray liners)

INSTRUCTIONS FOR USE:

1. Tray Liners can be sterilized according to the parameters detailed below.
2. Place liner(s) inside or under tray.
3. Place instruments in tray and wrap or close container according to manufacturer's instructions.


STERILIZATION:

Tray Liners are provided non-sterile and may be sterilized using the following cycle parameters:

Steam Sterilization				
Item Number	Cycle	Temperature	Minimum Time – Full Cycle	
TLP-012-018 TLP-020-025 TL-101-200 TL-105-250	TL-202-500 TL-203-500 TL-204-500	Pre-Vacuum	132°C (270°F)	4 minutes
TL-101-200 TL-105-250 TL-202-500	TL-203-500 TL-204-500	Gravity	121°C (250°F)	30 minutes
100% Ethylene Oxide (EtO)				
Item Number	Concentration	Temperature	Exposure Time	Humidity
TLP-012-018 TLP-020-025 TL-101-200 TL-105-250 TL-202-500 TL-203-500 TL-204-500	735 mg/L	55° C	60 minutes	50-70%

DISPOSAL:

Tray Liners should be disposed of after use; there are no special disposal instructions.

 MANUFACTURER: KEY SURGICAL LLC
8101 WALLACE ROAD / EDEN PRAIRIE, MN 55344 / USA
PH: 800.541.7995 KEYSURGICAL.COM



DO NOT RE-USE

LOT NUMBER

REF CATALOG NUMBER