

Utility Scissors

INTENDED USE

Key Surgical® Utility Scissors are all-purpose shears with a serrated stainless steel blade and plastic handle intended for Emergency Room, Orthopedics, and general hospital use.

PRECAUTIONS:

Sharp: Care should be taken during use.

STERILIZATION:

Provided non-sterile and have been validated for one-time sterilization efficacy at the following cycle parameters:

Steam Sterilization		
<i>Cycle Type</i>	<i>Temperature</i>	<i>Minimum Time – Full Cycle</i>
Pre-Vacuum	132° C (270° F)	4 minutes
Gravity	121° C (250° F)	30 minutes

It remains the responsibility of the processor to ensure that the processing, as actually performed using equipment, materials and personnel in the processing facility, achieves the desired result. This requires verification and/or validation and routine monitoring of the process.

DISPOSAL:

Requires replacing if there are any signs of deterioration or loss of functionality. Dispose according to facility protocol.



KEY SURGICAL LLC
8101 WALLACE ROAD / EDEN PRAIRIE, MN 55344 USA
(US) TEL: 800.541.7995 (UK) TEL: +44(0)1628 810626
(EU) TEL: +49 4363 905900



MANUFACTURER
FOR US: MANUFACTURED FOR



CATALOG
NUMBER



LOT
NUMBER



CAUTION

 **STERIS**®