

Vascular Loops

EN - INTENDED USE: Key Surgical® Vascular Loops are a disposable, single-use medical device intended for transient term (less than 60 minutes) use in surgical procedures requiring retraction, identification, or occlusion of peripheral vessels, nerves, ureters, or tendons. The Vascular loops are not intended for use in direct contact with the central nervous system.

PRECAUTIONS: This device is single use only; reuse may lead to patient cross contamination and/or device failure. Do not re-sterilize. Re-sterilization of the device may result in loss of product functionality.

CAUTION: Prior to use visually inspect packaging. Do not use if damaged, opened or breached as sterility cannot be guaranteed.

CONTRAINDICATIONS: There are no known contraindications associated with the Vascular Loops.

INSTRUCTIONS FOR USE:

1. Remove vascular loops from pouch.
2. Remove and discard clip.
3. Vascular loops may be single- or double-looped and clamped at the site or may be clamped to vascular surgical drapes.

DISPOSAL: Upon completion of procedure, dispose of both used and unused loops according to facility protocol. Do not reuse.

 KEY SURGICAL LLC
8101 WALLACE ROAD / EDEN PRAIRIE, MN 55344 USA
(US) TEL: 800.541.7995 (UK) TEL: +44(0)1628 810626
(EU) TEL: +49 4363 905900

 MANUFACTURER
FOR US: MANUFACTURED FOR

REF

CATALOG
NUMBER

LOT

LOT
NUMBER



CONSULT
INSTRUCTIONS
FOR USE

STERILE **R**
STERILIZED USING IRRADIATION



DO NOT USE IF PACKAGE
IS DAMAGED



CAUTION



DO NOT RESTERILIZE



DO NOT RE-USE



USE-BY DATE

MD

MEDICAL DEVICE

UDI

UNIQUE DEVICE IDENTIFIER



SINGLE STERILE BARRIER SYSTEM



SINGLE STERILE BARRIER SYSTEM
WITH PROTECTIVE PACKAGING OUTSIDE



STERIS®