

**INTENDED USE:**

Key Surgical® Instrument Sleeves are designed to protect the tips and edges of surgical instruments when not in use and during sterilization.

**CONTRAINDICATIONS:**


There are no known contraindications associated with the Instrument Sleeves.

**INSTRUCTIONS FOR USE:**


1. The instrument sleeves do not need to be cleaned or disinfected before use. Hold the sleeve flat with Key Surgical logo facing up.



2. Flip the sleeve top over bottom so the logo faces down and toward you at the bottom. Fold sleeve at perforation inserting into triangle slit.

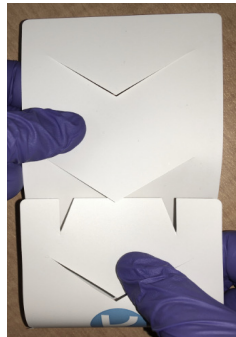

3. Fold the bottom corners at perforation inward to create a pocket.

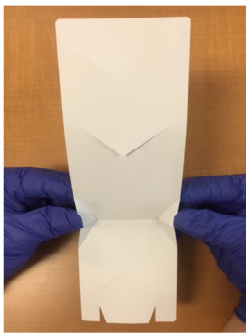

4. Slide instrument into top perforated triangle slit and weave through to bottom triangle slit and into bottom of sleeve.



5. Instrument sleeves are single use and can be sterilized only once. The sleeves should be disposed of after use.

**STERILIZATION:**

The Instrument Sleeves are provided non-sterile. They have been validated for steam sterilization at the following cycle parameters:

<i>Cycle Type</i>	<i>Temperature</i>	<i>Minimum Time – Full Cycle</i>
Gravity	121° C (250° F)	30 minutes
Pre-Vacuum	132° C (270° F)	4 minutes
Pre-Vacuum	134° C (273° F)	3 minutes

They have been validated for the following STERRAD® Systems and cycles:

<i>STERRAD® System and Cycle within the United States</i>	<i>STERRAD® System and Cycle outside United States</i>
STERRAD® 50	STERRAD® 50
STERRAD® 200	STERRAD® 200 Short cycle
STERRAD® 100S	STERRAD® 100S Short cycle
STERRAD® NX Standard cycle	STERRAD® NX Standard cycle
STERRAD® NX Advanced cycle	STERRAD® NX Advanced cycle
STERRAD® 100NX Standard cycle	STERRAD® 100NX Standard cycle

**STORAGE:**

There are no special storage instructions for the instrument sleeves.

**DISPOSAL:**

There are no special disposal instructions.