

Pipe Cleaner

INTENDED USE:

Key Surgical[®] Pipe Cleaners are designed to be used as cleaning verification of surgical instruments prior to sterilization.

CONTRAINDICATIONS:

There are no known contraindications and/or adverse effects.

INSTRUCTIONS FOR USE:

1. Cut pipe cleaners to length.
2. Protect cut ends by bending the end over and twisting.
3. If desired, double and twist the pipe cleaners together to increase diameter.
4. Insert the pipe cleaner clockwise into the instrument.

DISPOSAL:

Pipe Cleaners should be disposed of after use. Dispose of product following facility policy for disposal.



MANUFACTURER: KEY SURGICAL LLC
8101 WALLACE ROAD / EDEN PRAIRIE, MN 55344 / USA
PH: 800.541.7995 KEYSURGICAL.COM



CATALOG
NUMBER



MDSS GMBH
SCHIFFGRABEN 41, D-30175 / HANNOVER, GERMANY



CONSULT
INSTRUCTIONS
FOR USE



DO NOT RE-USE



BROC-3080, Rev F, 10/2018