

## Scope Baskets

### INTENDED USE:

Key Surgical® Scope Baskets are intended to protect scopes during sterilization, transport and storage. They are constructed of stainless steel and silicon.

### CONTRAINDICATIONS:

There are no known contraindications and/or adverse effects.

### PRECAUTIONS:

- Visually inspect prior to use. Check for loose wires or burrs that may puncture or tear sterilization wrap or pouch. Make sure all latches and handles are secure and in working order.

### PREPARATION:

Scope baskets are reusable and must be cleaned and disinfected prior to initial use and subsequent reuse.

### POINT OF USE CARE:

Clean Scope Basket as soon as possible after use. If cleaning must be delayed, immerse in an enzymatic solution or water to prevent drying and encrustation of surgical soil. Avoid prolonged exposure to saline to minimize the chance of corrosion. Remove excessive soil with a disposable wipe.

### MANUAL CLEANING:

- Pre-rinse under cold tap water for one (1) minute to remove gross debris
- Soak for a minimum of two (2) minutes in a pH neutral detergent, prepared in accordance with the manufacturer's instructions for use.
- Rinse under cold tap water for one (1) minute.
- Ultrasonically clean for a minimum of five (5) minutes in a neutral pH detergent, prepared in accordance with the manufacturer's instructions for use.
- Rinse under cold tap water for one (1) minute.

### AUTOMATED CLEANING:

It may be necessary to manually clean prior to automated processing to improve the removal of adherent soil. Follow the previous instructions for manual cleaning.

- Run the automatic wash cycle – minimum cycle parameters:
  - 1 minute cold pre-rinse
  - 5 minute enzyme wash at 43° C minimum temperature
  - 1 minute cold rinse
  - 7 minute dry at 90° C minimum temperature

### CLEANING INSPECTION:

Visually inspect before sterilization or storage to ensure the complete removal of soil from surfaces. If soil is still present, re-clean the Scope Basket.

### PACKAGING:

- Identify scope basket that will be used.
- Place scope in scope basket. Balance contents with uniformity within the basket and arrange to allow adequate sterilization exposure.
- Sterilize basket according to the parameters detailed below.

### STERILIZATION:

Follow the instructions for use of the sterilization packaging manufacturer, the enclosed device manufacturer and the sterilizer manufacturer.

Scope baskets are provided non-sterile and have been validated for sterilization according to applicable international process standards and guidance for the following methods and parameters:

Cycle Type (Steam)	Temperature	Minimum Time – Full Cycle
Gravity	121° C (250° F)	30 minutes
Pre-Vacuum	132° C (270° F)	4 minutes

STERIZONE® System	STERIZONE® VP4
-------------------	----------------

100% Ethylene Oxide (EtO)	
Concentration	725 mg/L
Temperature	55° C
Exposure Time	60 minutes
Humidity	50-80%

STERRAD® System and Cycle within the United States	STERRAD® System and Cycle outside United States
STERRAD® 50	STERRAD® 50
STERRAD® 200	STERRAD® 200 Short cycle
STERRAD® 100S	STERRAD® 100S Short cycle
STERRAD® NX Advanced cycle	STERRAD® NX Advanced cycle
STERRAD® NX Standard cycle	STERRAD® NX Standard cycle
STERRAD® 100NX Standard cycle	STERRAD® 100NX Standard cycle

It remains the responsibility of the processor to ensure that the processing, as actually performed using equipment, materials and personnel in the processing facility, achieves the desired result. This requires verification and/or validation and routine monitoring of the process.

### STORAGE:


- After sterilization, move wrapped set to sterile storage area for storage or transport to point of use location.
- Do not handle roughly or drop.
- Return scope basket to sterile processing department after set has been used.

### DISPOSAL:


Scope Baskets have a life span and will require replacing if there are any signs of deterioration or loss of functionality. Dispose of product following facility policy for disposal.


**MANUFACTURER: KEY SURGICAL LLC**  
 8101 WALLACE ROAD / EDEN PRAIRIE, MN 55344 / USA  
 PH: 800.541.7995 KEYSURGICAL.COM


 LOT NUMBER

 CONSULT INSTRUCTIONS FOR USE

 CAUTION



**MDSS GMBH**  
 SCHIFFGRABEN 41, D-30175 / HANNOVER, GERMANY

 REF CATALOG NUMBER