



Scope Brushes (Stainless Steel Handle, Single Use)

INTENDED USE:

Key Surgical® Scope Brushes (BR-3030, BR-3035, BR-3040, BR-3410, and BR-3420) are intended to be used in the manual cleaning of endoscopes. The brushes are used to clean the inside of the endoscope channel(s) and loosen soil present.

CONTRAINDICATIONS:

Do not use cleaning brushes for cytological tissue sampling or any other diagnostic or therapeutic purposes. Patient injury, cross-contamination and equipment damage may occur. Do not use these brushes for any purpose other than their intended use.

INSTRUCTIONS FOR USE:

- 1. Ensure brush is the proper diameter and length for the endoscope channel it is being used to clean.
2. Advance the brush in short strokes to avoid kinking of the handle.
3. Scope Cleaning Brushes should not be coiled in loops smaller than 10" (25cm).
4. To remove gross soil from the endoscope after brushing, flush the channel with water until the soil is removed. For specific endoscope cleaning instructions, refer to the endoscope manufacturer's IFU.

PREPARATION:

Brushes do not need to be cleaned or disinfected prior to use. The brushes can however, be put through a washer if desired.

INSPECTION:

Brushes should be inspected and replaced if worn, frayed, bent or otherwise damaged. Worn or damaged bristles and handles are ineffective in cleaning, and may damage the endoscope.

STERILIZATION:

This sterilization is intended for one time use of brushes in a sterile setting, and is not for brushes that have already been used.

New, unused brushes are provided non-sterile and have been validated for sterilization efficacy using steam sterilization according to the following parameters:

Table with 3 columns: Cycle Type, Temperature, Minimum Time - Full Cycle. Rows include Pre-Vacuum (132° C, 4 minutes) and Gravity (121° C, 30 minutes).

STORAGE:

Dispose of brush after use.

DISPOSAL:

There are no special disposal instructions.

MANUFACTURER: KEY SURGICAL LLC
8101 WALLACE ROAD / EDEN PRAIRIE, MN 55344 / USA
PH: 800.541.7995 KEYSURGICAL.COM

REF CATALOG NUMBER

LOT LOT NUMBER

EC REP MDSS GMBH SCHIFFGRABEN 41, D-30175 / HANNOVER, GERMANY

