



Disposable Corner Protectors

INTENDED USE:

Key Surgical® Tray Corner Protectors are intended for use with surgical trays to prevent the tearing of the sterilization wrap and to protect the corners of the trays from damage.

CONTRAINDICATIONS:

There are no known contraindications and/or adverse effects associated with the corner protectors.

INSTRUCTIONS FOR USE:

The corner protectors do not need to be cleaned or disinfected prior to use.

1. Place the corner protectors on the top and bottom corners of the tray before wrapping with sterilization wrap. Once in place, wrap the tray according to standard wrapping procedures.
2. The corner protectors can be sterilized according to the parameters detailed below


STERILIZATION:

Appropriate sterilant contact will occur on the surface of the corner protectors as well as the tray surface under the corner protectors:

<i>Cycle Type</i>	<i>Temperature</i>	<i>Minimum Time – Full Cycle</i>
Pre-Vacuum	132° C (270° F)	4 minutes
Gravity	121° C (250° F)	30 minutes

DISPOSAL:

The corner protectors are designed for one-time use and should be disposed of after use. There are no special disposal instructions.

 **MANUFACTURER: KEY SURGICAL LLC**
 8101 WALLACE ROAD / EDEN PRAIRIE, MN 55344 / USA
 PH: 800.541.7995 KEYSURGICAL.COM

REF CATALOG NUMBER

EC REP MDSS GMBH
 SCHIFFGRABEN 41, D-30175 / HANNOVER, GERMANY



LOT LOT NUMBER

 **CONSULT INSTRUCTIONS FOR USE**

 **DO NOT RE-USE**