

Write-On Tags

INTENDED USE:

Key Surgical® Write-On Tags are used to identify the status of or communicate special instructions for instruments, devices and other equipment.

INSTRUCTIONS FOR USE:

Write-On Tags are provided non-sterile and are for single use only. Write-On Tags can be written on with a lab marker containing ink that is waterproof, alcohol resistant and is compatible with steam and gas plasma sterilization. Secure the tag to the equipment using a cable tie (Key Surgical part number: RT-100T) or to the outside of the package.

Validation of tag compatibility with facility chemistries and washer/disinfector cycles should be undertaken by the end user.

STERILIZATION:

Write-On Tags are provided non-sterile and have been validated for sterilization inside and outside of sterilization containment systems according to applicable international process standards and guidance for the following methods and parameters:


Cycle Type (Steam)	Temperature	Minimum Time – Full Cycle
Gravity	121° C (250° F)	30 minutes
Pre-Vacuum	132° C (270° F)	4 minutes
Pre-Vacuum	134° C (273° F)	3 minutes

STERRAD® System and Cycle within the United States	STERRAD® System and Cycle outside United States
STERRAD® 50	STERRAD® 50
STERRAD® 200	STERRAD® 200 Short cycle
STERRAD® 100S	STERRAD® 100S Short cycle
STERRAD® NX Advanced cycle	STERRAD® NX Advanced cycle
STERRAD® NX Standard cycle	STERRAD® NX Standard cycle
STERRAD® 100NX Standard cycle	STERRAD® 100NX Standard cycle

It remains the responsibility of the processor to ensure that the processing, as actually performed using equipment, materials and personnel in the processing facility, achieves the desired result. This requires verification and/or validation and routine monitoring of the process.

DISPOSAL:

Write-On Tags should be disposed of after use in accordance with facility policy for disposal.

 MANUFACTURER: KEY SURGICAL LLC
8101 WALLACE ROAD / EDEN PRAIRIE, MN 55344 / USA
PH: 800.541.7995 KEYSURGICAL.COM



MDSS GMBH
SCHIFFGRABEN 41, D-30175 / HANNOVER, GERMANY



DO NOT RE-USE